

FFP


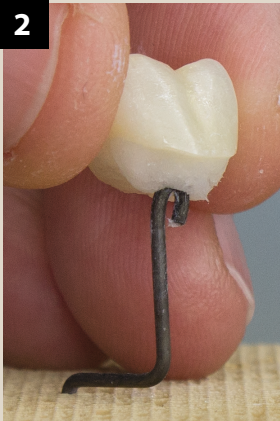
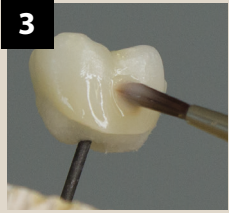
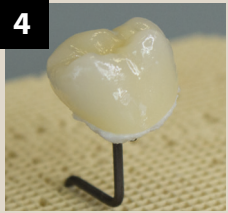
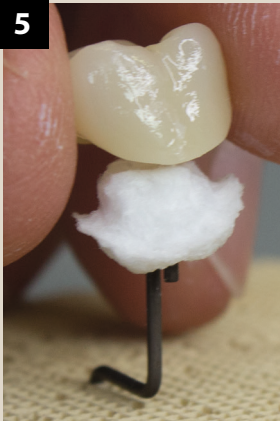
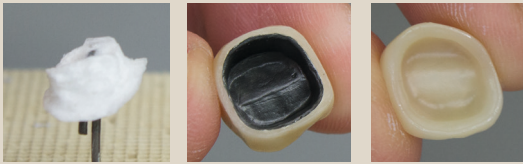

- more safety -
the small syringe with a big effect

excellence
line

Fix Firing Paste



The Fix Firing Paste is a ready-to-use paste for the fabrication of individual firing trays. With this refractory material, inlays, onlays, veneers, crowns and bridges can be securely positioned on pins or directly fixed to the firing tray.

<p>1</p>  <p>Apply ready-to-use fixing paste directly into the ceramic restoration</p>	<p>2</p>  <ul style="list-style-type: none"> - fix on the retaining pin - for veneers/inlays directly on the back surface - Direct fixation on the firing tray on the firing tray is possible, <u>in this case please fill the restorations a little higher.</u> 	
<p>3</p>  <p>further work steps like layering, staining ...</p>	<p>4</p>  <p>firing</p>	<p>5</p>   <p>After ceramic firing, simply lift restorations off the firing cotton die.</p> <ul style="list-style-type: none"> - easy to remove - no blasting or scratching out necessary 

Area of application / properties:

- Stabilisation of difficult-to-fix ceramic works during the firing process in the ceramic furnace
- easy application due to ready-to-use syringe
- no discolouration of the ceramic after firing
- easy to remove, no blasting or scraping necessary
- no extension of the pre-drying time in the ceramic furnace necessary

Content / Syringe: 12ml

Fix Firing Paste 1 syringe / 12ml Art.: 399-1000
Fix Firing Paste (5 pcs.) 5 syringes with 12ml each Art.: 399-1005